

Taking a
step forward
starts with
seeing where
you are

ONETOUCH[®]
every touch is a step forward[™]



Getting started
with the
OneTouch Reveal[®] app

ColourSure[™]
TECHNOLOGY


 **Bluetooth**[®]






The OneTouch Reveal[®] app changes the way you see your blood sugar


- Sync data, seamlessly, from your **OneTouch Verio Flex[™] meter** directly to your smartphone
- Draws a timeline of important blood sugar events and activities
- **ColourSure[™] technology** transforms data into quick visual snapshots that connect your blood sugar with food, insulin and activity
- Helps you see and share progress with your family, friends, or your trusted care network via email, or text, or during appointments

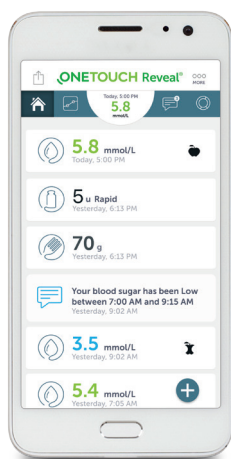

Timeline

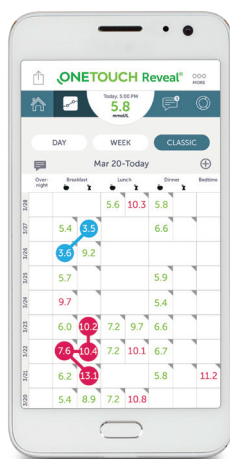

Logbook

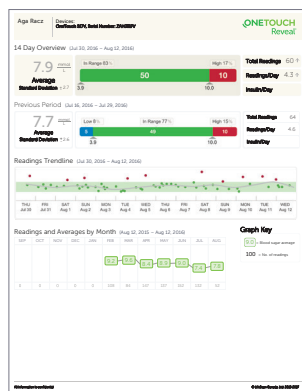

Progress Reports

Let's get started...


1. Download the free OneTouch Reveal® app

- Visit the app store on your mobile device and search "OneTouch Reveal"



2. Activate Bluetooth® on your OneTouch Verio Flex™ meter

- Press the **OK** button to turn your meter on
- Press the **OK** and **A** buttons at the same time to activate Bluetooth®.

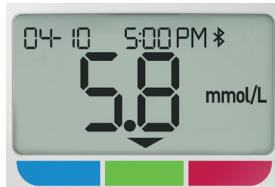


 **Bluetooth®**

Don't forget to also activate Bluetooth® on your mobile device.

3. Connect


- Open the OneTouch Reveal® app on your mobile device and follow the on-screen instructions
- When prompted, enter the PIN shown on your meter screen into your mobile device and tap **Pair**

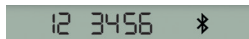


Example of a PIN number

Now you're ready to test and sync!

(Optional) Share data with your Care Team

- After setup, tap  **MORE** and select **Connections**
- Tap **Add a Clinic** and enter the **Clinic Code** provided by your Care Team and follow the rest of the on-screen instructions

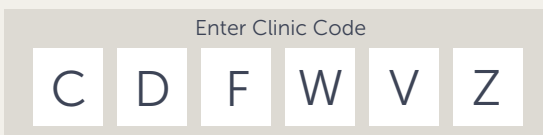

Discover more about the OneTouch Reveal® app



Visit OneTouch.ca



Call 1 800 663-5521



Or download your FREE OneTouch Reveal® app today



Apple, the Apple logo, and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play is a trademark of Google Inc. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by LifeScan Scotland is under licence. Other trademarks and trade names are those of their respective owners.

© 2017 LifeScan Canada. All rights reserved. NACO/VFX/0417/0303

ONETOUCH
every touch is a step forward™